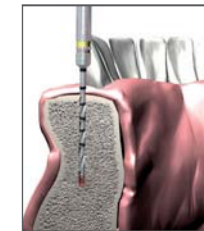
Bone Condensing

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In the case of Bone Condensing first the pilot bur produces a subdimensional bone cavity (take notice of diameter and depth of the drilling). In a narrow alveolar ridge situation the initial bur prevents the pilot bur from slipping.

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A sub-dimensional bone cavity for bone densities D3/D4 is then created using the conical expansion burs.

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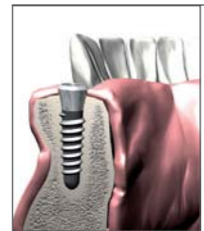
For Bone Condensing purposes of implant cavities of more than 2.7 mm diameter, use the expansion bur A2003 and the spreader A2005. For implant cavities exceeding 3.3 mm diameter, the expansion bur D2003 and the spreader D2005 must be used. After placement of the primary spreader (A2005 or D2005), the following spreaders in ascending order can be used without further drilling with pilot drills. In non-atrophied bone, the use of the spreaders leads to condensation of the bone and horizontally resorbed areas of the Alveolar ridge are expanded in addition. With the help of the ratchet the spreaders may be screwed in cautiously.

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The condensed and spread bone may now provide greater primary stability once an implant has been placed.

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- Ridge widening of the mandible by selective spreading and angle modulation of the implant bed
- Maximal conservation of existing bone substance
- Suitable for preliminary implant cavity preparation for all current implant systems
- Optimal in combination with Split-Control
- Minimally invasive alternative to osteotomes
- Controlled dilatation of horizontally resorbed bone
- Gentle densification of cancellous bone
- Suitable for bone resorption classes 1–3
- Provides high primary implant stability through bone condensing

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¹204=RA, 205=RA L ²Largest working part diameter in 1/10 mm

	Pilotbohrer Pilot burs Fresas piloto	Erweiterer Expansion burs Ensanchadores	Handgriff Handle Mango	Spreader Spreaders Spreader								
Fig.	B1001	E1001	A2003	D2003	CBH0G	Fig.	A2005	B2005	C2005	D2005	E2005	F2005
Shank ¹	206	206	205	205		Shank ¹						
Size ²	013	020	023	030		Size ²	027	029	031	033	035	040
Length mm	15,0	15,0	12,0	12,0		Length mm	12,0	12,0	12,0	12,0	12,0	12,0
▲			1,30	2,03		▲	1,70	1,91	2,12	2,33	2,54	3,06
▲			2,30	3,00		▲	2,70	2,90	3,10	3,30	3,50	4,00

¹204=RA, 206=RA XL, ²Largest working part diameter in 1/10 mm ▲ Minimal diameter

	Winkelaufsätze Angled attachments Cabezas del contrángulo		
Fig.	CBH00090	CBH00110	CBH00055
∠	90°	110°	55°
Size ²	027	029	027
	031	033	
	035	040	

²Largest working part diameter in 1/10 mm

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Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

Horizontal-Control Plus
Combined Axial Angle Adjustment System

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Horizontal-Control is an Axial Angle Adjustment System that, with its completely new and unique instruments, allows simple and minimally invasive ridge widening. This is achieved because of an angle modulation of the implant axis given by the actual bone supply by erecting and displacing of the cortical plate, at the same time condensing the bone structure. In this way, the mandible can be optimally prepared for the use of all current implant systems. An important advantage of this innovative method is the non-traumatic procedure with special angled hand instruments. For special cases, Horizontal-Control is ideal when combined with the bone spreading instruments from the Split-Control system.

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An indication for the use of Horizontal-Control is when "pear-shaped" reabsorbed or traumatized sections of the lower alveolar ridge exist where conventional implantation without preliminary Bone Management® would not be possible.

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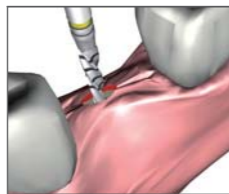
First, the laser marked twist drill produces a sub-dimensional pilot hole which determines the depth and direction of the implant to be inserted (take notice of diameter and depth of the drilling)

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
In the next step, the tapered expansion burs prepare a bone cavity appropriate for the subsequently used angle modulators.

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
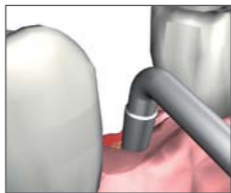
The controlled used and implementation of a series of tapered angle modulators with increasing diameters, widens the implant cavity in the spongiosa and carefully condenses the neighboring bone tissue (Use the the corresponding angled attachment by screwing it onto the handle). Successive alteration of the instrument alignment for angle modulation of the implant axis in the direction of the bone deficiency can erect detached segments, irrespective of the expansion of the alveolar ridge of up to 8 mm.

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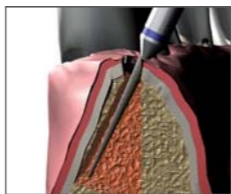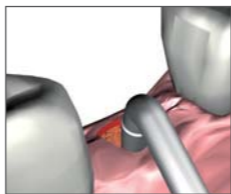
Should the existing jaw anatomy not allow enough expansion in the area of the implant cavity, a vertical osteotomy can be carried out (distal and mesial) internally without reflecting the periosteum. This causes a predetermined breaking point that allows the bone to break open in the following expansion process. The bone casing is slightly displaced outwards so that the alveolar ridge can be widened to the necessary width.

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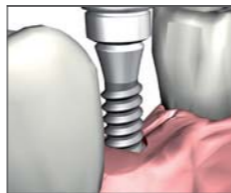The subsequently inserted implant holds the expanded cavity open and stabilizes the cortical walls.

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In special indications when the alveolar ridge has to be spread excessively, a combined application with the spreading instruments from the Split-Control kit is beneficial.

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In this case, before the angle modulation, the deployment of a series of spreaders with increasing diameters causes the bone to be additionally spread and to be condensed.

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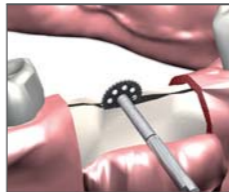
Split-Control is the minimally invasive alternative to osteotomes. Bone Spreading and Bone Condensing with special screw like instruments (spreaders) achieve a controlled and standardised dilatation of horizontally resorbed bone and a gentle densification of cancellous bone. The alveolar ridge can be optimally prepared for the subsequent process of implant insertion while maintaining the existing bone substance without complicated horizontal or vertical bone grafting.

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
To promote Bone Spreading a crestal relief cut with the osteotomy disk (between multi drilling) is implemented.

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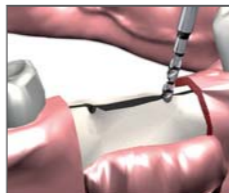
When Bone Spreading the initial bur prevents the pilot bur from slipping. Subsequently the pilot bur produces a subdimensional bone cavity (pay attention to depth of insertion).

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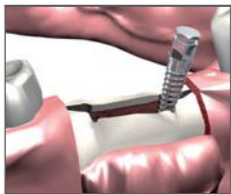The successive deployment of a series of spreaders with increasing diameters leads to Bone Spreading and condensing (order of diameter and codification of the instruments must be respected). With the help of the ratchet the spreaders may be rotated in cautiously. In bone density of 1–2 the drilling before deploying the spreader is prepared with the corresponding D1, D2 expansion bur.

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By having widened the implant cavity in this manner it is now possible to place the suitable implant. Furthermore bone rigidity, achieved by bone condensation, may lead to greater primary stability.

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